**LIST OF DISINFECTANTS  
DESINFEKTIONSMITTELLISTE  
LISTE DES DÉSINFECTANTS  
ELENCO DEI DISINFETTANTI  
LISTA DE DESINFECTANTES  
LISTA DE DESINFETANTES  
LIJST MET DESINFECTIEMIDDELEN  
LISTA ÖVER DESINFICERINGSMEDEL  
DESINFEKSJONSMIDDELLISTE  
LISTE OVER DESINFEKTIONSMIDLER  
DESINFIOINTIAINEIDEN LUETTELO  
ΚΑΤΑΛΟΓΟΣ ΑΠΟΛΥΜΑΝΤΙΚΩΝ  
POPIS DEZINFEKCIJSKIH SREDSTAVA  
LISTA ŚRODKÓW DO DEZYNFEKUJĄCE  
FERTŐTLENÍTŐSZEREK LISTÁJA  
СПИСЪК НА ДЕЗИНФЕКЦИОННИ ПРЕПАРАТИ  
SEZNAM DESINFEKČNÍCH PROSTŘEDKŮ**

23. Edition / 2021 – 05

The disinfectants listed below were **ONLY** tested on their material compatibility with:

- Haag-Streit measuring prisms
- Haag-Streit contact lenses

**WARNING!**

- The efficacy of the disinfectants was not tested on their correct disinfection effect on Haag-Streit measuring prisms and contact lenses.
- The sole purpose of the list below is to provide the user with a choice of suitable disinfectants which may be used for the validation of their own disinfection process.
- The efficacy of the disinfectants are determined by the disinfection manufacturers.
- The efficacy of the disinfectants on Haag-Streit measuring prisms and contact lenses must be guaranteed by the user or the reprocessing responsible person with the validation of their own disinfection process.
- Haag-Streit rejects all liability if any of the disinfectants mentioned in the list below is used without a proper validated cleaning and disinfection process.
- The disinfectants may only be used with the concentration mentioned on the list below.
- Any higher concentration may damage the measuring prisms or the contact lenses; in this case Haag-Streit rejects all liability.
- Haag-Streit rejects all liability if a disinfectant is used which does not appear on the list below.

**NOTE!**

Please refer to the Haag-Streit instructions for use "Cleaning and disinfection" for validated disinfection agents and for the correct cleaning and disinfection procedure for Haag-Streit measuring prisms and contact lenses.

**NOTE!**

Single use products and other accessories are the safest method to avoid the transmission of diseases. Haag-Streit recommends:

- **Tonosafe** – a disposable measuring prism for all Goldmann Applanation Tonometer.
- **Stery Cup** – a disposable shield for use with contact lenses 903, 903 L and Retina 145 L.

**List of disinfectants with good material compatibility with Haag-Streit products**

Product Types		Product Groups
Haag-Streit Contact Lenses	<sup>(1)</sup> 630 / 901 / 902 / 903 / 905 / 906 / 907 630 L / 903 L / 906 L / 907 L / Retina 145 L / CGAL / CGIL / CGPL / CGRL / CGVL 1210	<b>A</b> <b>B</b> <b>C</b>
Measuring prisms for Haag-Streit Applanation Tonometer		<b>D</b>

<sup>(1)</sup> Also valid for variant „S“

Disinfectant	Tested Concentration	Product group authorized for the disinfectant	Free of aldehyde	Containing aldehyde	Comments
NaOCl (Sodium Hypochlorite)	0,0525%	A / B / D	●		
NaOCl (Sodium Hypochlorite)	2,5%	A / B / D	●		max. 100 cycles of 1h
NaOCl (Sodium Hypochlorite)	10%	D	●		max. 100 cycles of 1h
NaOH (Sodium Hydroxide)	1M (mol / l)	A / D	●		max. 100 cycles of 1h
H <sub>2</sub> O <sub>2</sub> (Hydrogen Peroxide)	3%	A / B / C / D	●		
Acrylan®	<sup>(2)</sup>	B / C / D	●		
Almyrol®	4%	A / B / C / D	●		
Bodedex forte	1%	A / D	●		
Bomix plus	2%	A / D	●		
Cidex® plus	3,5%	D		●	
Cidex® OPA	0,55%	A / B / D		●	
Dakin's Solution	<sup>(2)</sup>	A / D	●		
Deconex 53 Instrument	2%	B / C / D	●		
Deconex 53 Plus	4%	B / C / D	●		
Endo Septol FF	5%	B / C / D	●		
Gigasept® AF	4%	D	●		
Gigasept® AF forte	5%	D	●		
Gigasept® FF NEW	8%	A / B / C / D		●	
Gigasept PAA concentrate	2%	A / D	●		
Jiaen 6% "Yoshida"	0,05%	A/D	●		
Korsorex basic	5%	A / D		●	
Korsorex extra	4%	A / D		●	
Mucocit®-T	6%	A / B / C / D	●		
NU-CIDEX®	<sup>(2)</sup>	A / D	●		
PeraSafe®	<sup>(2)</sup>	A / B / C / D	●		
Peraxylens®	<sup>(2)</sup>	A / B / C / D	●		
Perfektan® active	2%	D	●		
Perfektan® TB	4%	A / B / C / D	●		
Rapicide® OPA/28	100%	A / B / D		●	
Sekusept® Activ	2%	B / C / D	●		

Sekusept® forte S	3%	A / B / C / D	•	
Sekusept® PLUS	4%	A / B / C / D	•	
Stabimed®	2%	A / B / C / D	•	
Sterihyde® L	2%	D	•	
Tristel DUO	(2)	A / B / C / D	•	
Tristel Sporicidal Wipes	(2)	A / B / C / D	•	

(2) Activated solution according to the instructions of the manufacturer

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**HAAG-STREIT AG**